SUPERACRYL® PLUS

CS

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SUPERACRYL® PLUS

Heat-curing denture base acrylic resin

en

Characteristics:

A modern variant of the methacrylate basal resin. It is a methylmethacrylate polymer in powder form and methylmethacrylate monomer as a liquid. SUPERACRYL® PLUS is supplied in five shades O (colourless), U, V, Z, X (with fibres).

Indications:

- Bases of complete removable dentures.
- Bases of partial removable dentures.
- Manufacturing of orthodontic appliances.
- Indirect rebasing of removable dentures.

Contraindications: Not described.

Adverse reaction: Potential methacrylate sensitivity, especially at the allergic people.

Properties:

- The shortened time of polymer dissolution.
- The prolonged interval of the dough plasticity.
- The high bending strength.
- The low water solubility and absorbability.
- Easy to handle and polish.
- Cadmium free.
- Conforms to ISO 20795-1.

Processing:The preparation of dough:

The recommended mixing ratio is 22 g powder and 10 ml liquid, what represents in the volume ratio the value of about 3:1. The mixed material is left in the closed vessel about 12 - 16 minutes at the temperature of 23 °C. The dough is ready for further process at the moment when it stops to be sticky. The plasticity interval is about 10 minutes at the temperature of 23 °C.

Maturing time: 12 – 16 min

The plasticity interval: 10 min

Total working time: 25 min

Further on, it is proceeded by using the classic flasking in a plaster mould, and/or the modelling dough is processed by means of the so-called „free hand“ (at the partially removable substitutions) with the following polymerization in the pressure polymerization apparatus.

Polymerization methods:

- Immerse the flask into the cold water.
- Temper up to 70°C within 30 minutes.
- Keep the temperature during the next 30 minutes.
- Increase the temperature on 100°C within 30 minutes.
- Keep the temperature for the further 30 minutes.

The total polymerization time is 2 hours.

Warning:

Liquid: H225 Highly flammable liquid and vapour. H315 Causes skin irritation. H335 May cause respiratory irritation. H317 May cause an allergic skin reaction. H410 Very toxic to aquatic life with long lasting effects.

P210 Keep away from heat/sparks/open flames/hot surfaces. – No smoking. P262 Do not get in eyes, on skin, or on clothing. P270 Do not eat, drink or smoke when using this product.

Waste material disposal:

The residue of unused product and contaminated packaging material should be transferred to a company authorized to dispose of waste materials. Observe your local legal regulations.

Storage:

Store in a dry and dark place at 5 °C to 25 °C, in a well sealed container. Store the package in a well ventilated area, away from heat sources. Smoking is prohibited.

Shelf-life: 4 years

Packaging:

powder: 500 g in 5 shades O (colourless), U, V, Z, X (with fibres).

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