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## Nabelklemme

<b>DE</b>	Nabelklemme - Gebrauchshinweis	2
<b>EN</b>	Navel clamp - Instructions for Use	2
<b>BG</b>	Пълна клампа - Инструкции за употреба	3
<b>CS</b>	Pupěčnicková svorka - Návod k obsluze	3
<b>DA</b>	Navleklemme - Brugsanvisning	4
<b>EL</b>	Ομφαλόδεμα - Οδηγίες χρήσεως	4
<b>ES</b>	Pinza de cordón umbilical - Instrucciones de uso	5
<b>ET</b>	Nabaklamber - Kasutusjuhend	5
<b>FI</b>	Navan puristin - Käyttöohje	6
<b>FR</b>	Clamp ombilical - Mode d'emploi	6
<b>HR</b>	Stezaljka za pupkovinu - Uputa za upotrebu	7
<b>HU</b>	Köldökcsat - Használati utasítás	7
<b>IT</b>	Morsetto ombelicale - Istruzioni d'uso	8
<b>KO</b>	배꼽 폐색기 - 사용 설명서	8
<b>LT</b>	Virkštelės spaustukas - Naudojimo instrukcijos	9
<b>LV</b>	Nabassaites skava - Lietošanas norādes	9
<b>NL</b>	Navelklem - Gebruiksaanwijzing	10
<b>NO</b>	Navleklemme - Bruksanvisning	10
<b>PL</b>	Zaciskacz do pępowiny - Instrukcja obsługi	11
<b>PT</b>	clamp umbilical - Instruções de utilização	11
<b>RO</b>	Clemă ombilicală - Instrucțiuni de utilizare	12
<b>RU</b>	Пуповинный зажим - Инструкция по применению	12
<b>SK</b>	Pupočnicková svorka - Návod na Použitie	13
<b>SL</b>	Sponka za popkovino - Navodila za uporabo	13
<b>SV</b>	Navelklämma - Bruksanvisning	14
<b>TH</b>	อุปกรณ์หนีบสายสะดือ - คำแนะนำในการใช้งาน	14
<b>TR</b>	Göbek klempci - Kullanım için talimatlar	15
<b>ZH</b>	脐带夹 货号 - 使用说明	15

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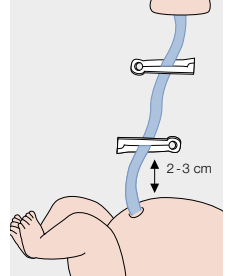
# EN: Instructions for Use - Navel clamp

## Designation of purpose

The Navel clamp is used during the birth of children. It is used to clamp the umbilical cord before the cord is cut.

## Product specification

The navel clamp is made of high-quality plastic and has a safety lock that cannot be opened after it has been engaged once. The navel clamp is in a blister, single sterile packed and only suitable for single use.

## Product handling

The cord should be clamped about 2 cm above the navel ring with the navel clamp. The umbilical cord should also be clamped in the direction of the placenta. It is important that the navel clamps are firmly closed and audibly click into place. If the clamp is closed properly, it cannot be opened without tools. Before the baby is wrapped in cloths or dressed, the position of the clamp must be checked again. If the navel has already dried up so much, the navel clamp can be removed by dropping off the rest of the navel cord or opening the navel clamp with a suitable tool.

## Storage

The navel clamps should be stored at room temperature and avoid exposure to direct sunlight. Do not use the navel clamps after the expiry date has passed.

## Disposal

After use, the navel clamp can be disposed of in normal household waste.

## Safety instructions/warnings

- do not use if packaging is opened or damaged
- always ensure correct closure/engagement of the clamp
- Do not remove until the navel is completely dry
- Use only by qualified personnel

 Do not use if package is damaged

 Risk of contamination if reused

 SARSTEDT AG & Co. KG  
Sarstedtstr. 1  
D-51588 Nümbrecht  
www.sarstedt.com

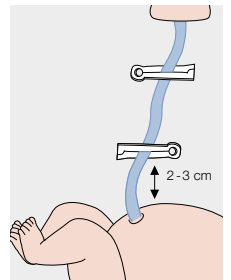
All serious incidents relating to the product shall be notified to the manufacturer and the competent national authority.

Store at room temperature

Technical modifications reserved

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