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STERILIT I oil spray JG600

Intended use

The STERILIT oil spray JG600 is used for steam sterilizing processes validated acc. to EN ISO17665-1 and dry heat sterilization at temperatures up to 180 °C.

STERILIT is applied prior to sterilization.

Properties

- No impairment of the sterilization effect from the lubrication film
- Lubricating effect and corrosion protection
- Silicone free

Safe handling and preparation

- ▶ Apply prior to sterilization only.
- ▶ Store in a cool, dry place, away from light.
- ▶ Observe the shelf-life date.
- ▶ Prior to using STERILIT, clean articles to be lubricated according to the instructions for use.
- ▶ Spray can under pressure: Protect against direct sunlight and temperatures above 50 °C.
- ▶ Do not pierce or burn the container, even after use.
- ▶ Do not spray near flames or glowing objects. Keep away from sources of ignition.
- ▶ Do not smoke.
- ▶ Keep product away from children.
- ▶ Always ensure sufficient ventilation to prevent formation of explosive mixtures.
- ▶ Read, follow, and keep the instructions for use.

Application

WARNING

Hazardous for aquatic organisms, can cause long-term harmful effects in water!

- ▶ Do not release into the environment.
- ▶ Obtain special instructions/consult safety data sheet.

- ▶ Briefly lubricate open instrument joints prior to the functional inspection and sterilization.
- ▶ For sites that are difficult to access, insert attached plastic tubing.
- ▶ Apply the oil spray directly on the link area.
- ▶ Evenly spread the oil by opening and closing the instrument, repeatedly.
- ▶ Remove excess oil with a lint-free wipe.
- ▶ Limit hot-air temperature to 180 °C.
- ▶ Especially in the case of hot-air sterilization, oil only the joints. Wipe excess lubrication oil from the surfaces, otherwise brown discoloration may form on the instrument surface as a result of heating the instrument oil.

Technical Service

WARNING

Risk of injury and/or malfunction!

- ▶ Do not modify the product.

- ▶ For service and repairs, please contact your national B. Braun/Aesculap agency.
- Modifications carried out on medical technical equipment may result in loss of warranty rights and applicable licenses.

Service addresses

Aesculap Technischer Service
Am Aesculap-Platz
78532 Tuttlingen / Germany
Phone: +49 (7461) 95-1602
Fax: +49 (7461) 16-5621
E-Mail: ats@aesculap.de

Or in the US:

Attn. Aesculap Technical Services
615 Lambert Pointe Drive
Hazelwood
MO, 63042

Aesculap Repair Hotline
Phone: +1 (800) 214-3392
Fax: +1 (314) 895-4420

Other service addresses can be obtained from the address indicated above.

Disposal

- ▶ Adhere to national regulations when disposing of or recycling the product, its components and its packaging!

Distributor in the US/Contact in Canada for product information and complaints

3773 Corporate Parkway
Center Valley, PA, 18034,
USA

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