SpofaDental
A Kerr Company

1000 g

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DEVISE DESCRIPTION
Ceradent is a modelling baseplate wax (Type 2) of class I (soft). Ceradent is available in plates 170 x 90 mm.

INDICATIONS FOR USE
Ceradent is indicated for occlusion templates, occlusion registration, space maintainers for making individual impression trays, saddle weight reduction in removable skeleton prostheses.

CONTRAINDICATIONS
None known

WARNINGS
None known

PRECAUTIONS
1. Ensure that the material is not too hot when applied to patient's mouth.
2. This product is intended to be used by qualified dental professionals on a general patient population.

ADVERSE REACTIONS
None known

INSTRUCTIONS FOR USE
Plastify the plate of wax by dipping it in warm water or holding it over the flame of a burner. At this stage it is easy to mold without tearing into pieces. After the softened material to the gypsum model and adapt it to the proper shape for occlusal registration. The wax can be used in the patient's mouth.

CLEANING AND DISINFECTION INSTRUCTIONS
Wax models should be safely disinfected with an aqueous disinfectant solution

after rinsing with water. Refer to the manufacturer's instructions for contact time and disinfectant concentration.

STORAGE, SHELF LIFE AND DISPOSAL
Store in a well-ventilated place at temperatures between 5 - 25 °C. Refer to the outer package for expiration date. Do not use after expiration date. For proper disposal always follow your local and national regulations.

WIGLANCE
If an adverse medical event occurs, report it to the manufacturer and to the competent authority for that country.

DISCLAIMER
The instructions provided above have been validated by SpofaDental as being CAPABLE of preparing a medical device, including reusable ones. It remains the responsibility of the dental professional / user to ensure that the processing actually performed by equipment, materials and personnel is done per the instructions provided above in order to achieve the desired result. Any deviation by the dental professional / user from the instructions provided above will be done at your own risk; the SpofaDental will not be able to address any requests for refund or exchanges under guarantee of any products that have not been handled or processed in accordance with the above instructions.

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