CHARISMA® Opal

CHARISMA Opal

- GB Instructions for use
- CZ Návod k použití
- PL Instrukcja obsługi
- BR Instruções de uso
- HR Upute za uporabu
- TR Kullanma talimatı
- RU Инструкция по применению
- AE تطبيقات الاستخدام
- CN 使用說明

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Dentistry – Polymer-based restorative materials ISO 4049 (type 1, class 2, group 1)

Instructions for use GB

CHARISMA® Opal
Medical Device – for use by dental health care professionals only.

Indications:
• Filings of the classes I–IV according to Black • Shape and shade corrections of teeth (e.g. diastema closure, congenital defects) • Splitting of teeth loosened by trauma or periodontal disease • Restorations of primary teeth • Temporary repairs of porcelain and composite (in combination with an adequate repair-system) • Direct composite veneers • Core build-up

Contraindications:
Application to dentine close to the pulp (less than 1 mm). The use of this product is contraindicated in case of known or suspected allergies against components of this product. Do not use this product in case of known or suspected allergies against (meth)acrylate compounds.

Potential Side Effects:
This product or one of its components may in particular cases cause hypersensitive reactions.

Composition:
Barium Aluminum Boro Fluor Silicate Glass, BisGMA, TEGDMA, Silica, Titanium Dioxide, 2,2-Dimethoxy-2-phenylacetone, Aminobenzoicester, Camphorquinone, BHT, Fluorescent Pigments, Metallic Oxide Pigments, Organic Pigments. Charisma Opal contains approximately 58% filler by volume, its filler particle size is 0.02–2 µm.

Application Hints:
• Visually inspect for damage before use. Damaged products must not be used. • Composites should only be used at room temperature. • In the case of deep cavity preparations in close proximity to the pulp or a direct pulpal exposure, cover the dentine or the exposure locally with a calcium hydroxide liner before applying the adhesive. • Do not use eugenol containing materials in or around the cavity preparation being restored, because this may interfere with the proper curing of the composite. • Use adequate protection for syringes to prevent from any contamination. Do not re-use after contamination.

Product Use:
Select an appropriate shade or shade combination with the CHARISMA Opal shade guide prior to placing the rubber dam. This is important because teeth will significantly change colour when desiccated. If the tooth surface being treated is not to be instrumented by a diamond stone, carbide bur, drill or other abrasive instrument, the surface should be cleaned with pumice or a fluoride free polishing paste before restoration is placed. A non-sticking placement instrument is recommended.

1. The use of a rubber dam (ivory*) or similar is recommended to control contamination from moisture, blood and saliva.
2. Prepare the cavity according to generally accepted rules for the clinical composite application. Then clean and dry the cavity.

3. Prior to application of CHARISMA Opal into the cavity, pre-treatment of the tooth structure with an adequate enamel/dentine adhesive is mandatory (e.g. GLUMA® adhesives by Kulzer). Once treated with the adhesive, avoid contamination with saliva or blood. The use of a flowable composite material (e.g. Charisma® Flow by Kulzer) allows the creation of a smooth cavity floor and facilitates the adaptation of the filling material. When using a layer of a flowable material, apply it according to the specifications in the respective instructions for use and cure it separately.

4. Apply CHARISMA Opal to the cavity in thin layers (max. 1.5 mm) and adapt carefully to the tooth surface or cavity walls. In case of clear discoloration of the underlying tooth or surface being restored, it may be necessary to mask this by using a thin layer of one of the recommended CHARISMA Opal opaque shades first.

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