# FD 360

DE	Kunstlederreinigung und -pflege	CS	Čištění a péče o umělou kůži
EN	Cleaning and care of vinyl upholstery	SK	Čistenie a ošetrovanie umelej kože
FR	Nettoyage et entretien du simili-cuir	SL	Čistilno in negolvano sredstvo za umetno usnje
IT	Pulizia e cura dell'ecopelle	HR	Čišćenje i njega umjetne kože
ES	Limpieza y mantenimiento del cuero	HU	Műbőrtisztítás és -ápolás
PT	Limpeza e cuidado da pele artificial	EE	Kunstnaha puhastamine ja hooldus
NL	Reiniging en verzorging van kunstleder	LV	Mākslīgās ādas tīršana un kopšana
SV	Rengöring och vård av konstläder	LT	Priemonė dirbtinei odai valyti ir prižiūrėti
FI	Tekonahan puhdistus ja hoito	RU	Очистка и уход за искусственной кожей
NO	Rensing og behandling av kunstlær	BG	Почистване и поддържане на изкуствена кожа
DA	Kunstlæderrensning og -pleje	RO	Curățarea și întreținerea pielii sintetice
EL	Καθαρισμός και περιποίηση συνθετικού δέρματος	TR	Suni Deri Bakımı ve Temizliği
PL	Czyszczenie i pielęgnacja sztucznej skóry		

DÜRR DENTAL AG  
Höpfigheimer Straße 17  
74321 Bietigheim-Bissingen, Germany  
Tel.: +49 7142 705-0  
Fax: +49 7142 705-500  
info@duerr.de · www.duerr.de

 **DÜRR  
DENTAL**

Z.360-2015-10

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FD 360 cleans and cares for all seat covers made from artificial leather and is highly effective in maintaining the value of dental treatment chairs and other artificial leather covers. You can apply FD 360 with the special sponge supplied in the set to rapidly and thoroughly remove stubborn grime, stains, and discoloration, caused for example by non-fast dyestuffs in clothing. FD 360 cleans and cares intensively without leaving any residues; it provides a silky gloss finish.

**Composition:** FD 360 is based on a mixture of special tensides, silicone compounds and care components derived from avocado oil.

**Application:** Use FD 360 undiluted. Spray FD 360 onto the surfaces to be cleaned and then immediately wipe off with a cloth. For intensive cleaning rub FD 360 on with the special sponge, which achieves an even more thorough and deeper cleaning effect. Subsequently, wipe off the excess FD 360 with a dry cloth. Use FD 360 daily if required.

**Notes:** FD 360 can be used without any problems on all artificial leather surfaces. The material tolerability of other leather surfaces should be checked on a concealed surface. The special sponges should **not** be used on leather.

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