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Wirkungsspektrum: Bakterizid¹⁾, tuberkulocid, levurocid, begrenzt virucid activity (enveloped viruses, incl. HBV, HCV and HIV¹⁾, as well as non-encapsulated viruses, such as noro viruses). **List of VAH.** Tested in accordance with EN 14348 and EN 14476.

¹⁾ in accordance with the recommendations of the RKI (Robert Koch Institute) (Bundesgesundheitsbl. 47, 62-66, 2004).

Directions for use: Pour FD 366 sensitive into the spray bottle. Spray the surfaces and articles to be disinfected until fully wet or apply FD 366 sensitive to a cloth and wipe. The exposure time in accordance with VAH is 1 minute (see table in "DE"). For further information please also see product information on the Internet.

Caution: Flash point in accordance with DIN 51 755 is 33 °C. Maximum spread 50 ml/m².

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Vaccinia-Viren inkl. HBV, HCV, HIV^{1),2)}

30 Sek.

Kettering, Northants NN16 8PS, United Kingdom

Lot

+44 1536526740

DÜRR DENTAL (Produkts) UK Ltd.

14 Linnwell Way - Telford Way Industrial Estate

Kettering, Northants NN16 8PS, United Kingdom

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DÜRR DENTAL GERMANY

EN 14348, EN 14476.

EN FD 366 sensitive disinfection of sensitive surfaces

FD 366 sensitive is an aldehyde-free, ready-to-use solution for the rapid disinfection and cleaning of sensitive surfaces such as plastics and artificial leather coverings on treatment units, acrylic glass, polymerisation lamps, equipment covers, mixing bowls (including bowls from vacuum mixers), hose holders, spittoons, control panels

on treatment units or X-ray development equipment, etc.

FD 366 sensitive dries off quickly.

Composition:

100 g

of FD 366 sensitive contain

17 g

of 1-propanol,

0,25 g

of 50 %

alkyl-benzyl-dimethyl-ammonium chloride, auxiliary substances and water.

Spectrum of activity:

bactericidal,

tuberculocidal,

yeasticidal,

limited

virucidal activity (enveloped viruses, incl. HBV, HCV and HIV¹⁾, as well as non-encapsulated viruses, such as noro viruses). **List of VAH.** Tested in accordance with EN 14348 and EN 14476.

¹⁾ in accordance with the recommendations of the RKI (Robert Koch Institute) (Bundesgesundheitsbl. 47, 62-66, 2004).

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