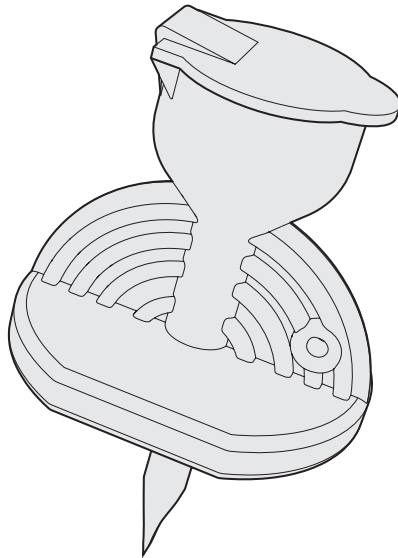
Mini-Spike®

B | BRAUN

B. Braun Melsungen AG
34209 Melsungen
Germany

www.bbraun.com

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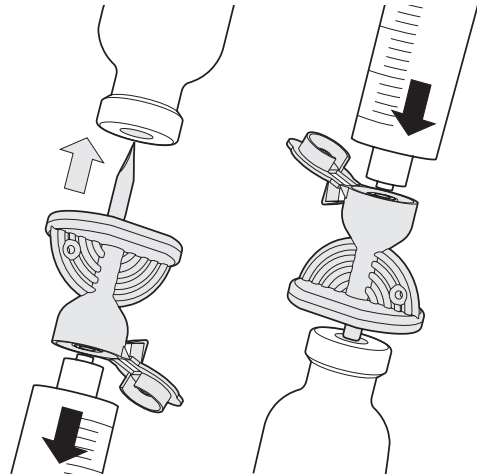
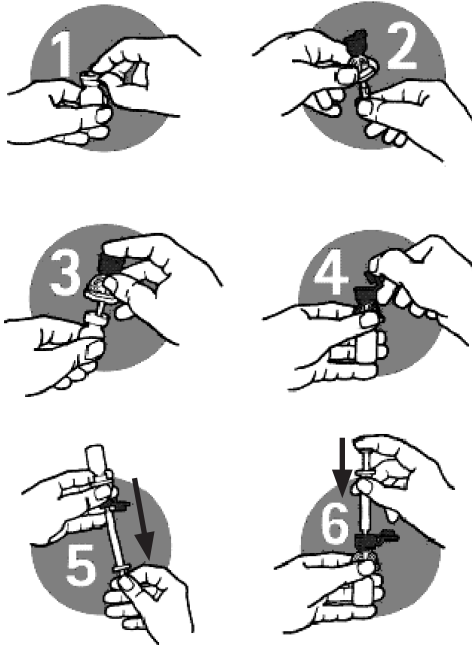
GB Instructions for Use

Application

- ① Remove protective cap from the medication container and disinfect the stopper.
- ② Remove protective cap from the spike.
- ③ Insert spike.
- ④ Open snap cap.
- ⑤ Connect syringe and aspirate required volume.
- ⑥ **Important:** Solution from a filled syringe into the container (Fig. 6) must only be injected right-side up. Risk of excess pressure and impaired deairing.

Materials used

Housing: ABS/SAN,
closing cap: PP,
protective cap and valve stamp: PE,
filter: acrylic copolymer on nylon body material,
valve disk: silicone

Duration of application

Please refer to the instructions for use of the medication as well as valid hygiene and safety regulations.

WARNINGS:

Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.

Filtration:

Due to the possible adsorption > 15 %, do not use spikes with liquid filters/particle filters (venting filters are noncritical) with: corticotropin, cosyntropin, cyanocobalamin, ergonovines, maleates, glucagon, insulin, vasopressin.

No information is available as regards interactions with medication other than those listed.

Substances causing tension cracks:

Various inorganic, surface-active substances and solvents, e.g. macrogol (polyethylene glycol) and/or benzyl alcohol, in medication or active substances, may lead to stress and tension cracks in plastics.

With these aggressive substances we recommend use of the spike for a single aspiration only.

Manufacturer disclaims any responsibility for possible consequences resulting from improper use.

Date of last revision March 2013

Indications and product variations

Removal or injection spike for multiple withdrawals from multi-dose vials

	A	B	C	D	E	*
Mini-Spike® (REF 4550242)	✓					green
Mini-Spike® Micro-Tip (REF 4550510)	✓		✓			green
Mini-Spike® V (REF 4550560)	✓				✓	green
Mini-Spike® Filter (REF 4550234)	✓			✓		blue
Mini-Spike® Filter Micro-Tip (REF 4550528)	✓		✓	✓		blue
Mini-Spike® Filter V (REF 4550579)	✓			✓	✓	blue
Mini-Spike® Chemo (REF 4550340)		✓		✓		red
Mini-Spike® Chemo Micro-Tip (REF 4550536)		✓	✓			red
Mini-Spike® Chemo V (REF 4550587)		✓		✓	✓	red

* Closing cap colour

A: Venting filter 0.45 µm

Protects the medication from contamination by bacteria.

B: Venting filter 0.2 µm

Protects the medication against contamination by bacteria and the user from contamination by aerosols.
Specially suited for use with toxic substances.

C: Micro tip

Reduces the residual volume.
Specially suited to small volume, expensive medications.

D: Liquid filter/particle filter 5 µm

Protects against contamination by particles.
Specially suited for use with medications in powder form.

E: Valve

The valve also closes upside down containers and is drip proof.

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	A	B	C	D	E	*
Mini-Spike® (REF 4550242)	✓					yeşil
Mini-Spike® Micro-Tip (REF 4550510)	✓		✓			yeşil
Mini-Spike® V (REF 4550560)	✓				✓	yeşil
Mini-Spike® Filter (REF 4550234)	✓			✓		mavi
Mini-Spike® Filter Micro-Tip (REF 4550528)	✓		✓	✓		mavi
Mini-Spike® Filter V (REF 4550579)	✓			✓	✓	mavi
Mini-Spike® Chemo (REF 4550340)		✓		✓		kırmızı
Mini-Spike® Chemo Micro-Tip (REF 4550536)		✓	✓			kırmızı
Mini-Spike® Chemo V (REF 4550587)		✓		✓	✓	kırmızı

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AU For sterilization type see primary packaging

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CA Distributed by: **B. Braun Medical Inc.**
Bethlehem, PA USA 18018-3524

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ID Imported by:
PT. B. Braun Medical Indonesia
Jakarta, Indonesia.

KR B. Braun Korea Co., Ltd.
13th Floor, POSCO Center Bldg. No. 892, Daechi 4-Dong,
Kangnam-Ku
Seoul 135-777, KOREA

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US Rx only
Pictorials are for reference only
Distributed by: **B. Braun Medical Inc.**
Bethlehem, PA USA 18018-3524

Manufacturing site:
B. Braun Melsungen AG
34209 Melsungen, Germany

CE 0123

1213

12410454