<b>ESPE</b>
<b>RelyX™ Temp NE</b>
<b>Temporary Cement – Zinc Oxide Non-Eugenol</b>
<b>Цинкоксидный безэвгенольный цемент для временной фиксации</b>
<b>Цимент за временно центриране без евгенол</b>
<b>Cink-oksíd cement bez eugenola za privremeno cementiranje protetskih nadomjestaka</b>
<b>Eugenol nem tartalmazó ideiglenes cinkoxid cement</b>
<b>Tempzasowasy cement na bazie tlenku cynku bez zawartosci eugenolu</b>
<b>Ciment temporar pe bază de oxid de zinc, fără eugenol</b>
<b>Dočasny cement na báze oxidu zinočnatého bez eugenolu</b>
<b>Zink oksidov začasni cement brez eugenola na osnovi cinkovega oksida</b>
<b>Provizorni cement bez eugenolu</b>
<b>Geçici Siman-Çinko Oksit-Öjenolsüz</b>
<b>Eugenooli mittesaldav ajutine tsinkoksiit-tement</b>
<b>Pagaidu cements – cinka oksids, nesatur eigenolu</b>
<b>Laikinas cinko oksido cementas be eugenolio</b>
<b>Цинкооксидний безэвгенольный цемент для тимчасової фіксації</b>

Instructions for Use
Instrukcija po primenieniu
Указания за употреба
Upute za upotrebu
Használati utasítás
Instrukcja
Instrukcija užicia
Instrucțiuni de Utilizare
Návod na použitie

<span></span>	<span></span>
<b>3M Deutschland GmbH</b> Dental Products Carl-Schurz-Str. 1 41453 Nessau – Germany	<span></span>
<b>3M ESPE</b> Dental Products 2510 Conway Avenue St. Paul, MN 55144-1000 U.S.A.	<span></span>
3M, ESPE and RelyX are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 2012, 3M. All rights reserved.	<span></span>

<span></span>	<span></span>
<b>CE</b>	<b>0123</b>
<span></span>	<span></span>
<b>ENGLISH</b>	<span></span>

**Description of product**
RelyX™ Temp NE is an eugenol-free self-curing cement to be used for temporary fixation of dental restorations. By adding Vaseline the final hardness of the cement can be reduced for easier removal of bridges, for example. RelyX™ Temp NE **does not affect** the polymerization of filling and luting composites. RelyX™ Temp NE is especially suitable for use on patients with sensitivity to eugenol.

ⓘⓂ These Instructions for Use must be kept for the duration of product use.

#### Fields of application

- Temporary fixation of provisional restorations.
- Temporary fixation of crowns, bridges, inlays, and onlays.
- Fixation of crowns and bridges on implant abutments

#### Precautionary Measures

#### For Patients

The catalyst paste contains substances that may cause an allergic reaction in case of skin contact in certain individuals. Avoid use of this product in patients with known allergies to colophony and its derivatives. If prolonged contact with oral tissue occurs, flush with large amounts of water. If an allergic reaction occurs, seek medical attention as needed, remove the product and discontinue future use.

#### For Dental Personnel

The catalyst paste contains substances that may cause an allergic reaction in case of skin contact in certain individuals. To reduce the risk of an allergic response, avoid exposure to this material, in particular to the uncured product. If skin contact occurs, wash skin with soap and water.

The use of protective gloves and a no-touch technique is recommended. If the catalyst paste contacts the gloves, remove and discard the gloves, wash hands immediately with soap and water and then re-glove.

If an allergic reaction occurs, seek medical attention as needed.

3M MSDSs can be obtained from www.mmm.com, or contact your local subsidiary.

#### Protection of the pulp

In case of deep cavities apply the protection of the pulp usually employed for temporary fixation.

#### Dispensing and mixing

- Use RelyX Temp NE at an ambient temperature of 18° to 23° C/64° to 73° F and a relative humidity of 45 to 55%.
- Dispense two equal lengths of base paste and catalyst onto a mixing pad.
  - Overdosing the catalyst will increase the bonding strength and prolong the working time.
  - Close the tubes immediately after use. Avoid interchanging the tube caps to prevent the possible hardening of the paste in the tube as a result of an unintentional carry-over of the pastes.

- The final hardness of the cement can be reduced by adding Vaseline in order to ensure easier removal, e.g., from final bridges. This simultaneously extends the working time.
  - Dosing: Add a maximum of 42 mg (the size of a lentil) of Vaseline if a length of 1 cm of base paste (172 mg) and of catalyst (78 mg) is used.
- Use a metal or plastic spatula to thoroughly mix the pastes for 30 seconds until a homogeneous consistency is achieved.

#### Application

- Before cementing, clean the cavity surfaces with a water spray and then dry. Water and alcohol residues in the cavity accelerate the setting.
- Apply a thin layer of cement on the surface to be cemented, and place the dental restoration with slight pressure.

#### Times

The following times apply for an ambient temperature of 23° C/73° F:

	min:sec	with maximum amount of Vaseline min:sec
Mixing	0:30	0:30
Working after start of mixing	1:30	2:30
Setting after start of mixing	3:30	4:00

Setting is accelerated over an ambient temperature of 23° C/73° F and a relative humidity of over 55%.

#### Removal of excessive material

- Use an instrument to remove any excessive material after setting has occurred.

#### After Cementation

- Thoroughly examine the sulcus of the treated teeth and surrounding dentition. Remove any residual cement from the mouth.

#### Storage and shelf-life

Store the product at 15°–25° C/59°–77° F.

Do not use after the expiry date.

#### Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

#### Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH’s sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

#### Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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